

U-shaped opening toilet bath stool

U型開口衛生間洗浴坐凳

Model: LR-JB

Features:

- Using high-quality aluminum alloy with a wall thickness of 1.25mm, light weight, good toughness, and no rust
- The seat board and backrest are made of environmentally friendly materials, which are easy to clean, easy to dry, strong, etc.
- The handrails are made of environmentally friendly foam, which is waterproof, non-slip and comfortable to the touch
- Six levels of height can be adjusted, and it can be easily adjusted by pressing the marbles
- Suction cup foot pads, ordinary non-slip foot pads can be replaced
- The product is suitable for: the elderly, pregnant women and other people with mobility impairments
- 採用1.25mm壁厚的優質鋁合金，輕便、韌度好、不生鏽
- 坐板和靠背採用環保材質，易清潔、易幹、結實等
- 扶手採用環保泡棉，防水、防滑，手感舒適
- 六檔高度可以調節，按下彈珠即可輕鬆調節
- 吸盤腳墊，普通防滑腳墊可替換使用
- 產品適用於：老人、孕婦等行動不便者



Specifications

Product Model 產品型號	LR-JB
Product Material 產品材質	Aluminum alloy 鋁合金
Size 尺寸	49*43*70cm
Net weight 淨重	3.2kg
Packing size 包裝尺寸	79*59*21cm
Package weight 包裝重量	3.7kg

Instructions

1. Open the package and take out all accessories
 2. Especially important: the triangle on the sitting board must be underneath
 3. Align the other leg with the hole, put it down, put a washer, and screw on the short screw
 4. Put on two backrest tubes, pad gaskets, and screw on long screws
 5. Put on the backrest, pad the gasket, and screw on the short screws
 6. Insert the two armrests into the holes reserved for the leg tubes on both sides, and the installation is complete
1. 打開包裝取出所有配件
 2. 特別重要:坐板上的三角形一定要在下
 3. 另一根腳管對準孔位放下去,墊上墊片,擰上短螺絲
 4. 放上兩根靠背管,墊上墊片,擰上長螺絲
 5. 放上靠背,墊好墊片,擰上短螺絲
 6. 將兩根扶手分別插進兩側腳管預留的孔中,安裝便完成