

Aluminum Alloy Manual Wheelchair Portable

鋁合金手動輪椅輕便便攜

Model : LR-TA

Features:

- Widened seat design, fast folding in 5 seconds, more convenient travel
 - Lightweight and compact, there are 4 brakes at the front and rear, so you can use it with confidence
 - Concave-convex non-slip armrest ring, easy to use Convex shape more non-slip
 - Oxford fabric is easy to clean and easy to take care of
 - The aluminum alloy frame is sturdy and durable, and the dual-support frame has strong stability
-
- 加寬座位設計, 輕便小巧前後共4剎車, 方便護理, 放心使用
 - 快速折疊, 只為出行更便捷
 - 凹凸防滑扶手圈, 使用方便凹凸外形更防滑
 - 牛津布料方便清洗, 易打理
 - 鋁合金車架堅固耐用, 雙支持車架穩固性強



Specifications

Product model 產品型號	LR-TA
Product function 產品功能	portable manual wheelchair 手動輪椅輕便便攜
Material 材質	aluminum alloy + oxford cloth 鋁合金+牛津布
front tyre 前輪尺寸	6 inches
Rear tyre 後輪尺寸	20 inches
Net body weight 車身淨重	11.5kg
Product folding size 折疊尺寸	27*66*72CM
Product weight 產品重量	1.80kg
Packing Weight 包裝重量	13kg
Package Size 包裝尺寸	30*73*74CM

Instructions

◆輪椅展開和折疊方法:

1. 雙手握住把手, 將車後輪提起離開地面, 向兩側輕拉
2. 將左右車架分開 (靠墊上沿繃直), 向下輕壓坐墊兩側至輪椅展開
3. 將腳踏板放平, 系好腿帶
4. 用手提拉坐墊中間, 使輪椅車兩側大架慢慢向中間收攏
5. 按住左右扶手外側向內壓合即可

◆注意事項:

1. 向下壓坐墊時切勿將手指放入夾縫中, 以免受傷
2. 切勿在未剎住佇立剎車時乘坐輪椅, 以免輪椅滑動造成身體傷害
3. 切勿腳踩腳踏板上下輪椅, 謹防輪椅前傾造成身體傷害
4. 切勿抓握其他部位抬動輪椅, 以免造成輪椅損壞或身體傷害
5. 下坡時, 不能使用佇立剎車, 以免翻車造成人身傷害
6. 下車時請務必先剎住佇立剎車, 佇立剎車是為使用者上下輪椅或停靠時使用, 不能用來在行駛中作制動使用