

# Aluminum Alloy Four Legged Crutch Walking Aid

## 鋁合金四腳拐仗助行器

Model : LR-TP

### Features:

- 1 Made of ABS material, with 10 levels of adjustable height, can be fixed by rotating the ring
- It adopts thickened aluminum alloy tube body, which is light and durable
- Four-corner crutches, enhanced support, no kicks, no knocks, more at ease to use
- The foot pad is non-slip and wear-resistant, which enhances the friction on the ground and strengthens the stability
- 採用ABS材質，設有10檔可調高度，使用者可按照自己需求來調節高度
- 採用加厚鋁合金管身，具有輕便耐用的特性
- 旋轉圓環即可固定，使用不會產生搖晃
- 四角拐杖，增強支撐力，不打腳，不磕腳，用起來更放心
- 腳墊防滑耐磨，增強了地面的摩擦，加強了穩固性



### Specifications

Product model 產品型號	LR-TP
Product function 產品功能	four legged crutch walker 四腳拐仗助行器
Material 材質	aluminum alloy 鋁合金+ABS
gear 檔位	10檔可調節
Color 顏色	black/Silver 黑色/銀色
Product size 產品尺寸	72*20*17CM
Product weight 產品重量	600g
Package size 包裝尺寸	75*23*19CM
Packing weight 包裝重量	1.24kg

### Instructions

- ◆ 鬆開緊箍件
- ◆ 將彈珠移到合適孔位
- ◆ 確認彈珠是否扣住
- ◆ 擰緊緊箍件

#### ◆ 注意事項:

- 1) 使用前拐杖調節方法：按下彈鈕，上下調節確認合適高度後，將螺母鎖緊固定，方可使用
- 2) 檢查手把是否松脫，毛刺，謹防松脫及毛刺破損手掌而影響安全使用
- 3) 檢查防滑腳墊是否堅固平穩，若磨損應及時更換，斜滑的路面要特別小心
- 4) 拐杖是步行的輔助工具，請不要用於輔助工具以外的支撐物品之用
- 5) 拐杖不適宜登山運動和長時間支撐體重之用
- 6) 此拐杖為步行輔助器，請正確使用該產品，避免因不當使用造成意外傷害